ULTRASONIC AIR SALINIZER

Owner's Manual

Declaration of Conformity:

This device was manufactured and controlled by ISO 9001 Certified Management System and conform to all aspects of the agreed specification. This device complies with the essential requirements and provisions of Council Directive MDD 93/42/EEC for Medical Device Class I

EC REP Wellkang Ltd (www.CE-marking.eu)
29 Harley St., London W1G 9QR UK

Manufacturer: Halosense Inc

Address: 48 Knights Bridge Rd, London, ON, N6K 3R4, Canada

Technical Support: Halosense Inc. Ph: +1-519-641-7258 www.SalineTherapy.com

Congratulations on your purchase of the **SaltAir^{UV}- Home Salt Therapy** ultrasonic air salinizer!

SaltAir^{UV} is an ultrasonic particles dispenser using ultrasonic frequency vibration theory to transform the saline solution into 1-5 microns salt particles. These particles are able to penetrate deep into the lungs, to alveoli, and help to clear secretions, fight bacteria, viruses and fungus, reduce inflammation, improve bloodstream oxygenation and humidify the entire respiratory tract.

Salt therapy has been successfully used in the treatment of **chronic or acute respiratory diseases**, as well in prevention. It helps to reduce inflammation, unclog blockages, clean the respiratory system and improve breathing. It also prevents the development of respiratory diseases, and protects against cold and flu viruses by maintaining a good hygiene throughout the entire respiratory system.

Salt therapy is safe for all people and ages, including infants and pregnant women. It does not interact with any salt free diet, medication or medical condition including high blood pressure. The tiny particles of salt deposit on the respiratory mucosa and come off with mucus. Salt therapy can also be used for respiratory problems in pets.

SaltAir^{UV} air salinizer comes with an electric adaptor, instruction manual, measuring cup and **free three month supply of natural rock salt** for the saline solution. This device is easy to use. Just plug-in and use it over night. Place the device close to your bed, same level with your body, on a night table and enjoy the wonderful effect of natural salt therapy. The **SaltAir^{UV}** device is very quiet and will not disturb your sleep. For best results, we recommend continuous use

OPERATING INSTRUCTIONS

1. Place the device on a level, stable and dry surface where it will be used (nightstand) and remove the upper cover and the inside water cover. (Fig 1). Connect the AC adaptor to the base. (Fig 2)
2. In order to fill the water tank, use the measuring cup provided and fill the water tank up to 300ml with water. (Fig 3)
3. Use one measuring cup of water to dissolve $\frac{1}{2}$ **teaspoon** rock salt and pour the saline solution into the water tank, over the existing water, not exceeding the Max level. It is normal to have deposited impurities in the cup after dissolving the rock salt that is natural and unprocessed – please discard those. NEVER exceed the Max level! NEVER fill in water during operation! If the tap water is hard or contains chlorine it is better to use bottled water. (Fig 4)
4. Start as recommended with $\frac{1}{2}$ **teaspoon** of rock salt and increase the amount if needed, at the next refill. Do not exceed 1 teaspoon of rock salt for the whole water tank capacity but rather maximize your exposure to salt therapy.
5. Wipe dry the water tank using a paper towel. Place the water cover (doughnut shape) over the surface of the water and close the unit with the upper cover. (Fig 5)
6. Plug in the electric adaptor. (Fig 6)
7. Push the ON/OFF button. The power LED turns 'On' and the device will start working. The microscopic salt particles come out of the device through the top opening. **Mist is not very visible and this is normal!** Only a light air stream will be felt coming out of the device. (Fig 7)
8. **TIMER** – you can set the unit to work for 3h, 6h, 9h, ON, OFF when consecutively pressing the button. (Fig 8)
9. **LIGHT** – offers two UV light intensity for water treatment against bacteria or mold; you can use it at night during functioning or during the day when the unit is OFF. (Fig 9)

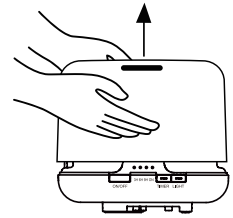
Fig 1

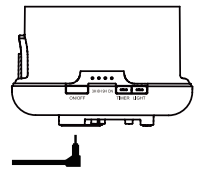
Fig 2

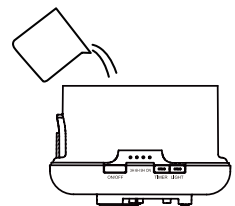
Fig 3

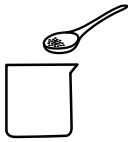
Fig 4

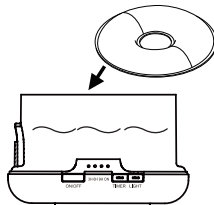
Fig 5

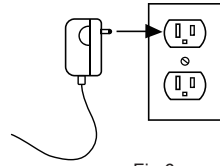
Fig 6

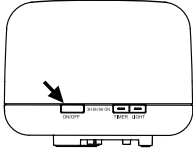
Fig 7

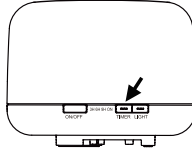
Fig 8





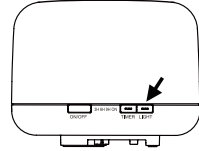
Fig 9

The device stops automatically when there is no more saline solution in the tank. **Expect the water tank to empty in about 5-9 nights of use.** To continue to use, refill the tank with saline solution.

Do not tilt or move the unit or remove the top covering after is filled or in use.


If there is no water in the water tank, pressing the ON/OFF will cause the unit to “beep” to remind you to add water. The unit will not work without water.

Product Details

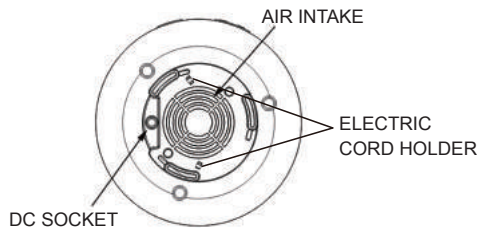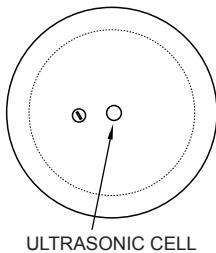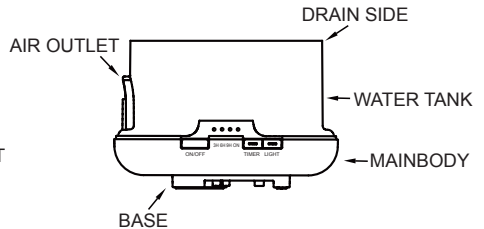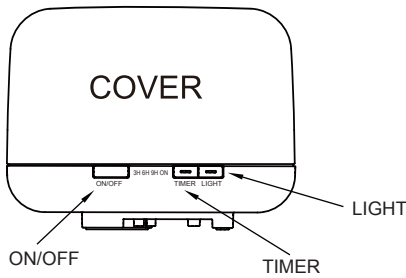INNER COVER

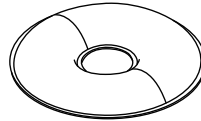WATER COVER

Maintenance, Cleaning and Storage

1. Unplug the device and remove the cover.
2. Pour out any remaining water on the DRAIN SIDE of the tank, assuring that no water gets into the AIR OUTLET. Do not immerse the unit under water! Clean the water tank at each refill using a soft cloth and clean water. Do not use petroleum, paint thinner or other chemicals for cleaning.
3. Gently clean the ultrasonic cell (the penny size white round piece on the bottom of the base), using a soft cloth or cotton bud and clean water. If there are calcareous deposits you can use 1 tablespoon of citric acid or 50 ml white cleaning vinegar in 200 ml water to clean the water tank and the ultrasonic cell. You can do this only every 2-3 months or when there are visible deposits.
4. Cover has an inner cover that can also be removed for cleaning. Find the grip and pull the inner cover out; after cleaning, insert first the air outlet opening and then press at the grip side until you hear the 'click'.

VERY IMPORTANT: Use only a soft cloth to clean the ultrasonic cell. Never touch it with sharp or hard objects. NEVER immerse the base into water and assure no water gets inside through the AIR OUTLET or other openings. DO NOT move the device from one place to another once it is filled. Always take the cover off and empty before moving it to another location.

5. For storage, clean the device carefully as instructed. After it is dry, put the device in its original packaging and keep it in a cool, dry place.

Transport and storage environment: -20 C to 55 C, humidity ≤95%, pressure 70-106 kPa.

Working environment: 5- 40 C, humidity 30-80%, pressure from 80kPa to 106kPa.

Technical data

Model	Input voltage	Power	Tank capacity	Noise
SaltAir ^{uv} – ST-UV01	DC 24V	13W	500 ml	<35dB
Adapter: Input 100-240V~ 50/60Hz, 300mA Output 24V, 500mA Adapter model NO.: SAA-240005WS-US or SAA-240005WS-EU				
Protection against electric shock		Class II (with adapter)		
Applied part		No applied part		
Mode of operation		Continuous operation		
Protection against harmful ingress of water or particulate matter		IPX0		
Note: Not intended to be sterilized. Not for use in an OXYGEN RICH ENVIRONMENT.				

Safety Cautions

- To reduce the risk of fire or electric shock, do not immerse this device in water; for assistance refer to qualified personnel only; never attempt to repair the device yourself.
- Adapter is part of the equipment. Put the device near the socket. If an extension cord is used it must be suitable to the power consumption of the device, otherwise overheating will occur.
- The adapter insulate the device from the mains supply. Do not use in a place where it is difficult to disconnect from the mains supply.
- Do not place this device close to other electric devices which may be damaged or cause electric hazard in case of an accidental saline solution spill.
- Do not fill the water tank with water over 40 °C to avoid discoloration or deformation.
- When the ambient temperature is below 0 °C pour out the water from the device.
- Use only the measuring cup provided to add water to the tank, never fill directly from the tap.
- Keep the device away from direct sunlight and heat sources, air conditioners or fans.
- The device must be placed on a hard surface to assure proper air flow underneath.
- Do not unplug with wet hand.
- If there is an accidental spill, unplug the device first.
- Do not touch the ultrasonic cell while the device is working.
- Do not touch adapter output when the device is working.
- Do not tilt or move the device after is filled or in use - it may cause water to get over electric parts and cause malfunction.
- Do not use for more than 10 hours at a time. Wait 60 minutes before restart to avoid damaging the ultrasonic cell.
- This device is not intended for use by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the device by a person responsible for their safety.
- Never allow it to be used by handicapped children or disabled individuals.
- Push the power button 'OFF' and unplug the adapter from the mains supply to safely terminate the operation.
- **Accidental leakage / unit tipped over:** turn OFF the power and remove cover; pour out any remaining water, tap dry the unit and let it dry for 24 hours before use.

TROUBLESHOOTING

If the **DEVICE** is not working correctly, please check the following causes:

TROUBLE	PROBABLE CAUSE	SOLUTION
DEVICE does not operate	Power cable not plugged in	Plug in power cable; disconnect and re-connect carefully
Switch OFF early	Not enough water in the tank	Add water to the tank
No mist or abnormal mist	Not enough water Too much water	Fill tank with saline solution Never fill tank over MAX line
	Ultrasonic cell needs cleaning	Please refer to maintenance instructions to clean the ultrasonic cell correctly
	Cover and inner cover not installed properly	Remove and re-install correctly, so vapors can be released freely
	Restricted air intake port on the base	Clean or free the air intake port on the base
Aerosol with unpleasant smell	New machine	Open the water tank and leave it for 12 hours in a cool, dry place
	Saline solution left in the tank for too long time, without use	Clean the water tank and make new saline solution

TROUBLE	PROBABLE CAUSE	SOLUTION
The power light is blinking and the DEVICE does not work	Saline solution got inside of the base	Clean the device and let it dry for 12 hours in a cool, dry place
Abnormal noise	The DEVICE is on unstable surface	Move the DEVICE to a stable, perfect level surface
	The saline solution got inside of the base over the fan	The DEVICE needs repair
Salt crystallized around midline or slowly leakage	The outside surface of the water tank remained wet or the cover was removed between refills.	Wipe the DEVICE dry after refill and do not remove the cover between refills.
Water leakage from DEVICE	Cover or inner cover not installed properly	Remove and re-install correctly, so vapors can be released freely
	Device has been tipped over	Unplug, pour any water left, wipe and let it dry for at least 24 hours
	Low temperature or high humidity	Under these conditions, the mist may condense quickly into water droplets. Assure proper air qualities.

WARRANTY AND DISCLAIMER

SaltAir^{UV} comes with one year manufacturer warranty. Halosense Inc. repairs or replaces equipment that proves to be defective during the warranty period. Warranty includes parts and labor.

Manufacturer will provide circuit diagrams, component part lists, descriptions, calibration instructions to assist service in parts repair.

The warranty provided herein does not cover equipment subjected to abuse, misuse, accident, alteration, neglect, or unauthorized repair or installation. Halosense Inc determines if the cause of defect is due to a manufacturing malfunction or customer misuse. **Note:** *The device has to be used only with our provided rock salt for the warranty to be in effect. Using a different kind of salt will void the warranty, as it can affect negatively the lifetime of the device and the health results!*

EXCEPT AS SPECIFIED HEREIN, HALOSENSE INC. MAKES NO WARRANTIES, EXPRESSED OR IMPLIED, AND SPECIFICALLY DISCLAIMS ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. CUSTOMER'S RIGHT TO RECOVER DAMAGES CAUSED BY FAULT OR NEGLIGENCE ON THE PART OF HALOSENSE INC. WILL BE LIMITED TO THE AMOUNT PAID BY THE CUSTOMER. HALOSENSE INC. WILL NOT BE LIABLE FOR DAMAGES RESULTING FROM LOSS OF DATA, PROFITS, USE OF PRODUCTS, OR INCIDENTAL AND CONSEQUENTIAL DAMAGES.

Symbols that we used here:

Signs	Notes on the signs
	Class II equipment
	Indoor use only
	Consult instructions for use
	International Recycling Symbol shows that these materials can be changed into new products to prevent waste of potential useful materials.
	Symbol for CE Mark. This symbol certifies that a product has met European Union consumer safety, health or environmental requirements.
	Where you see either symbol on any of our electrical products, batteries or packaging, it indicates that the relevant electrical product or battery should not be disposed of as general household waste in Europe. To ensure the correct waste treatment of the product and battery, please dispose them in accordance with any applicable local laws or requirements for disposal of electrical equipment/batteries. In so doing, you will help to conserve natural resources and improve standards of environmental protection in treatment and disposal of electrical waste.

EMC declaration

This product needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided, and this unit can be affected by portable and mobile RF communications equipment.

- 1) Do not use a mobile phone or other devices that emit electromagnetic fields, near the unit. This may result in incorrect operation of the unit.
- 2) Caution: This unit has been thoroughly tested and inspected to assure proper performance and operation!
- 3) Caution: this machine should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, this machine should be observed to verify normal operation in the configuration in which it will be used
- 4) Warning: The use of ACCESSORIES, transducers and cables other than those specified, with the exception of transducers and cables sold by the MANUFACTURER of the COMPRESSOR NEBULIZER as replacement parts for internal components, may result in increased EMISSIONS or decreased IMMUNITY of the ME EQUIPMENT or ME SYSTEM.

Guidance and manufacture's declaration – electromagnetic emission		
The ULTRASONIC AIR SALINIZER is intended for use in the electromagnetic environment specified below. The customer or the user of the ULTRASONIC AIR SALINIZER should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The ULTRASONIC AIR SALINIZER use RF energy only for its internal function. Its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	The ULTRASONIC AIR SALINIZER is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacture's declaration – electromagnetic immunity			
The ULTRASONIC AIR SALINIZER is intended for use in the electromagnetic environment specified below. The customer or the user of ULTRASONIC AIR SALINIZER should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4	±2 kV for power supply lines	±2kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s)	±1 kV differential mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U_T (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles <5% U_T (>95% dip in U_T) for 5 sec	<5% U_T (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles <5% U_T (>95% dip in U_T) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the ULTRASONIC AIR SALINIZER requires continued operation during power mains interruptions, it is recommended that the ULTRASONIC AIR SALINIZER be powered from an uninterruptible power supply or a battery.
Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	3A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

Guidance and manufacture's declaration – electromagnetic immunity

The ULTRASONIC AIR SALINIZER is intended for use in the electromagnetic environment specified below. The customer or the user of ULTRASONIC AIR SALINIZER should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the ULTRASONIC AIR SALINIZER, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d=1.167\sqrt{P}$ $d=1.167\sqrt{P}$ 80 MHz to 800 MHz $d=2.333\sqrt{P}$ 800 MHz to 2.5 GHz
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the ULTRASONIC AIR SALINIZER is used exceeds the applicable RF compliance level above, the ULTRASONIC AIR SALINIZER should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the SALINIZER.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the ULTRASONIC AIR SALINIZER

The ULTRASONIC AIR SALINIZER is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the ULTRASONIC AIR SALINIZER can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the ULTRASONIC AIR SALINIZER as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 KHz to 80 MHz $d=1.167\sqrt{P}$	80 MHz to 800 MHz $d=1.167\sqrt{P}$	800 MHz to 2.5 GHz $d=2.333\sqrt{P}$
0.01	0.117	0.117	0.233
0.1	0.369	0.369	0.738
1	1.167	1.167	2.333
10	3.689	3.689	7.379
100	11.667	11.667	23.333

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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